# User and maintenance manual and CE certification

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### GENERAL DESCRIPTION OF THE CERAMIC BOX

The BOX is made of a galvanised steel plate painted with highly resistant epoxy paints equipped with a 346 Watt induction motor with internal air recycle through an activated carbon outlet filter. Internally it is provided with a wedge adjustable in height and width and a PVC tank to collect the residual products. A pedal activates an air jet which comes out from an articulated arm, to clean or to cool metals. In the space below there are 2 slanted and adjustable straticolor arm rests. The system is frontally closed by a tempered crystal sash with a steel frame painted as requested by the client: this crystal is protected by another glass layer. The case is lighted by two high performance lamps. The supplied filters have the following features: 9,200 cm<sup>2</sup> nominal filter surface, 20 micron filter nominal capacity. The panel comprises the following switches:

1 lamp switch	16A
1 suction switch	16A
1 micromotor switch	16A
1 air damper and 1 gauge	

The BOX is powered by a 220Volt/50Hz voltage.

The total electrical input amounts to 390 watt/h (micromotor excluded).

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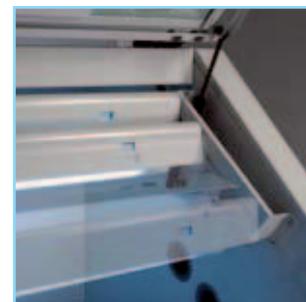
*The BOX comes with an one-year period warranty which does not cover any damage or misuse caused by users.*

*Use the BOX carefully and in a proper way, avoiding any improper or faulty installation breaching this warranty or safety rules in compliance with its manufacturing standards. Follow this manual very carefully, concerning the proper use and maintenance of the BOX appliance. Before using the appliance, any operator must be perfectly trained on its functions and technical specifications.*

*The appliance comes with filters; the package also contains: 1 user and maintenance manual and CE certification.*

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## LIGHTENING SYSTEM

The appliance comes equipped with 2 lamps, adjustable in height and slant pressing their ends with both hands.

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## PLUG SOCKET AND AIR INTAKE CONNECTION

On the back there is a power plug connection provided with a little drawer containing 2 delayed 10A fuses and an 8/6 air intake connection (see picture beside).

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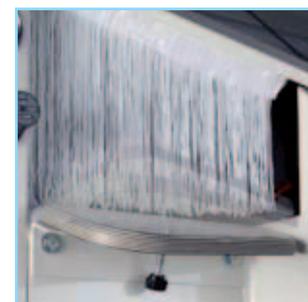
## ADJUSTABLE WRIST REST

Wrist rests slant can be adjusted by turning the knob located below it.

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## PROTECTION GLASS

To replace or to clean the protection glass open the door, unscrew the screws and extract the metal plate (see picture).

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Extract carefully the glass (see picture) and replace or clean it. Then reassemble repeating the previous steps in reverse order.

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## ADJUSTABLE WEDGE

Wedge adjustable in height and depth thanks to a rack rail.

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## ENTRANCE

The arm positioning is facilitated by a bevel edge on the front side of the hood.

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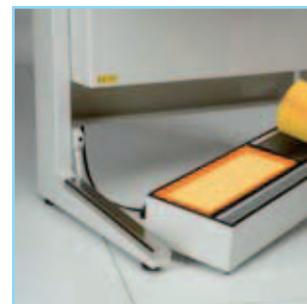
## SUCTION FILTER

To replace the lower filter, located near the appliance legs, lower the levers (see picture 1 and 2), extract the container and replace the filter (see picture 3 and 4).

To reassemble, repeat the previous steps in reverse order.

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## TANK

The central tank is extractable, as shown in the picture above.

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## FILTER ACTIVATED CARBONS

To replace the activated carbons located in the internal filter, unscrew the knob as shown in the picture above and lift the grate. To reassemble, repeat the steps in reverse order.

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## GAUGE

The appliance is provided with a gauge and a knob to control the air flow.

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**IRIDE INTERNATIONAL S.R.L.**

## **DECLARATION OF CONFORMITY**

We, IRIDE INTERNATIONAL S.r.l. – furniture for dentist offices and dental laboratories, with registered headquarters in Via San Giuseppe 5 - 33082 Azzano Decimo - Località Tiezzo - PN:

**HEREBY DECLARE UNDER OUR OWN RESPONSIBILITY THAT THE PRODUCT "CERAMIC BOX" TO WHICH THIS DECLARATION RELATES WERE DESIGNED AND BUILT IN EXCLUSIVE MANUFACTURING IN CONFORMITY WITH THE FOLLOWING HARMONISED STANDARDS:**

**IEC EN 60601-1(IEC 65-5) of 1991 and its 1994 amendments**

**IEC EN 60204-1(IEC 44/5 and 44/6)**

**AND THE REQUIREMENTS OF THE DIRECTIVE:**

**2006/42/CE**

**2006/95/CE**

**2004/108/CE**

**IRIDE INTERNATIONAL S.R.L.**

**IRIDE INTERNATIONAL s.r.l.**

Via S. Giuseppe 5 | 33082 Azzano Decimo - PN

tel. +39 0434 633242 | fax +39 0434 632546

welcome@irideinternational.com | www.irideinternational.com